medartis®

PRECISION IN FIXATION

INSTRUCTION / ANLEITUNG

Assembly / Disassembly Montage / Demontage

APTUS®
MODUS®

APTUS

Dismantling Multi-Part Instruments Demontage mehrteiliger Instrumente

Art. No. Nr.	Description Bezeichnung	Reference to Art. No. Nr.*	Page Seite
A-2030	1.2–2.3 Depth Gauge 1.2–2.3 Tiefenmessgerät	A-2031	
A-2031	2.0–2.8 Depth Gauge 2.0–2.8 Tiefenmessgerät		3
A-2032	2.0/2.3 Depth Gauge 2.0/2.3 Tiefenmessgerät	A-2031	
A-2730	2.5 Depth Gauge 2.5 Tiefenmessgerät	A-2031	
A-2836	2.8 Depth Gauge 2.8 Tiefenmessgerät		4
A-2930	3.5/4.0 Depth Gauge 3.5/4.0 Tiefenmessgerät	A-2836	
A-2038	2.2/3.0 Drill Stop for A-3738, A-3838 2.2/3.0 Bohrstopp für A-3738, A-3838		5
A-2073	Cannulated Handle with Quick Connector, AO Handgriff kanüliert mit Schnellkupplung, AO		6
A-2096	Aiming Device Zielgerät		7–9
A-2791.10/20	Saw Guide 45°/90° Sägeführung 45°/90°		10
A-2791.30	Compression Element Kompressionselement		11
CONTAINER			
A-6000	Base Frame Modular Implant Container Grundrahmen modularer Implantatcontainer		12–13
A-04xx	Implant Container Radial Head 2.0 Implantatcontainer Radiuskopf 2.0	A-6000	
A-05xx	Implant Container Foot 2.0, 2.3, 2.8 Implantatcontainer Foot 2.0, 2.3, 2.8	A-6000	
A-07xx	Implant Container Radius 2.5 Implantatcontainer Radius 2.5	A-6000	
A-08xx	Implant Container Hand 1.2, 1.5, 2.0, 2.3 Implantatcontainer Hand 1.2, 1.5, 2.0, 2.3	A-6000	

* We refer to specimen instructions for similar type instruments

MODUS

Dismantling Multi-Part Instruments

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
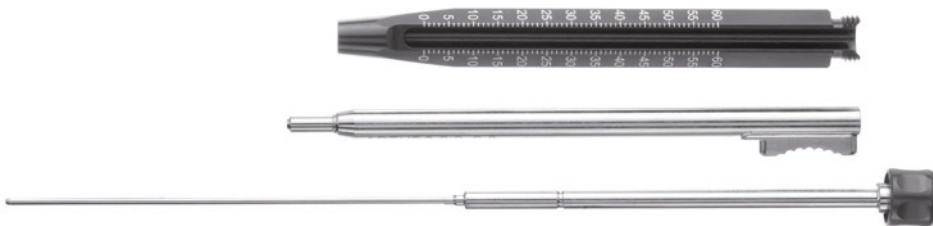
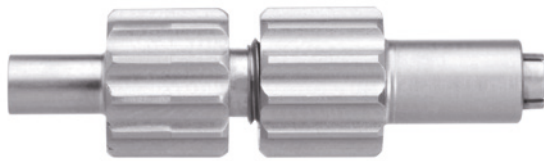
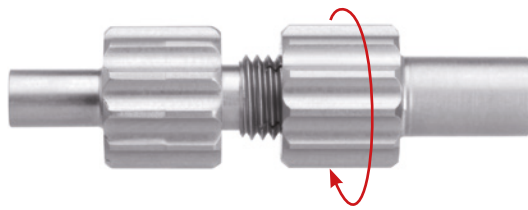
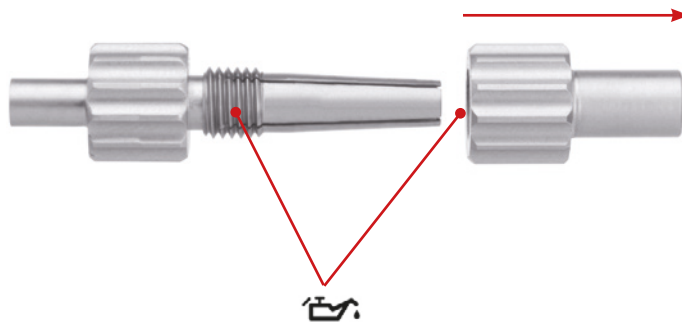
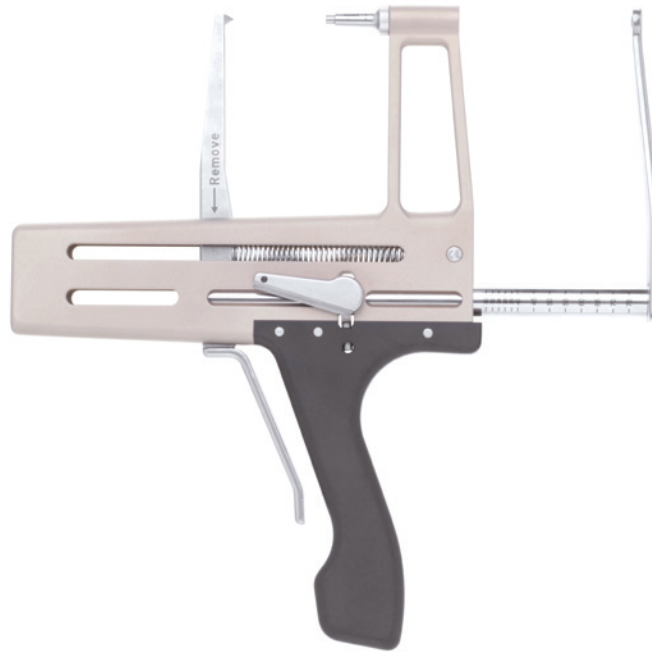
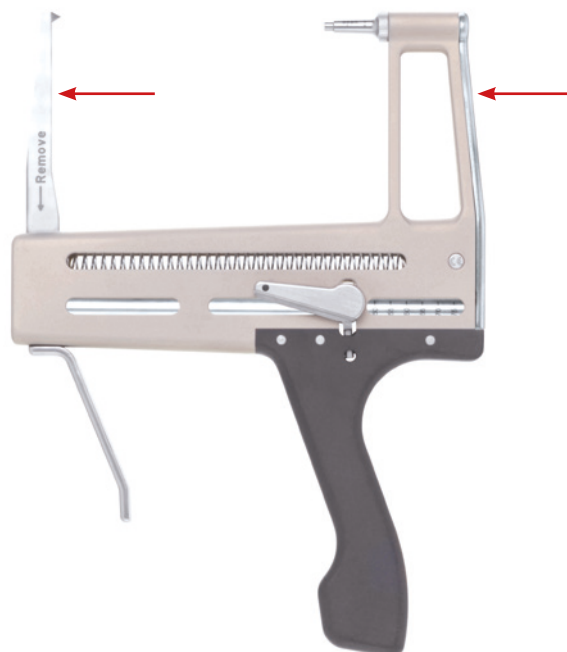
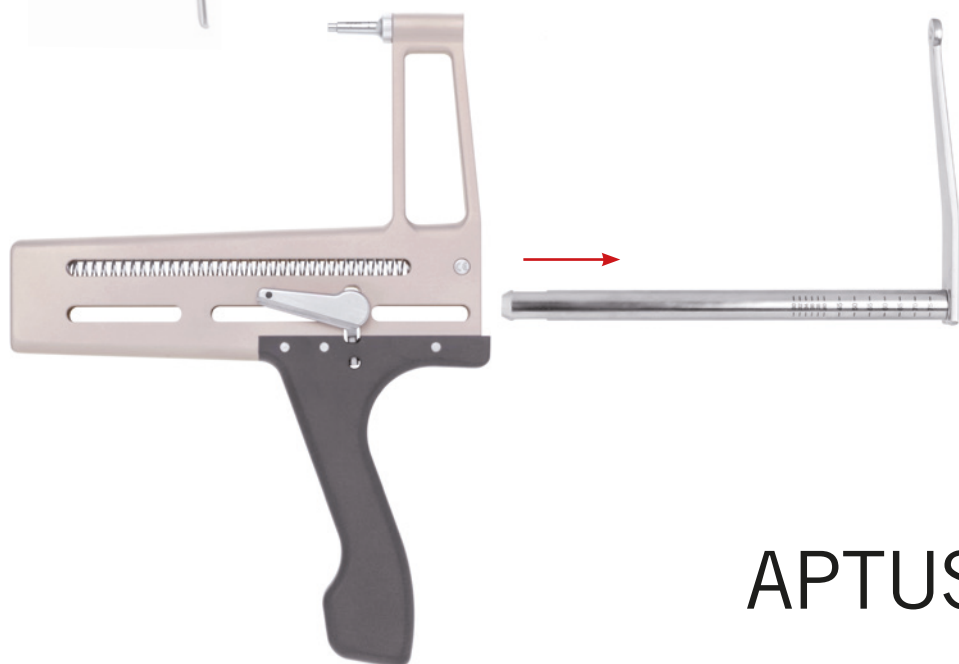
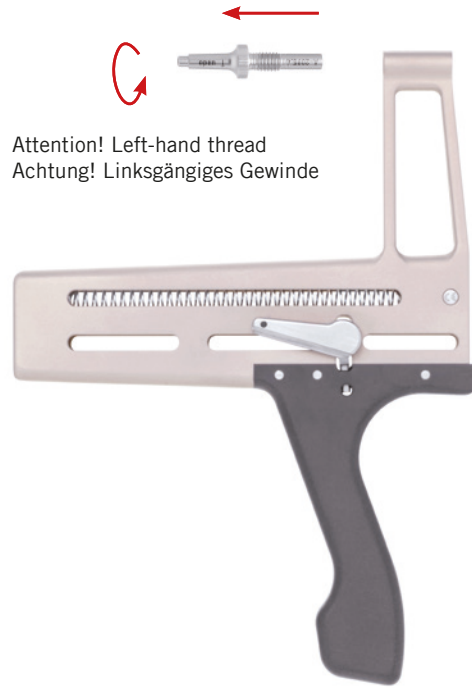
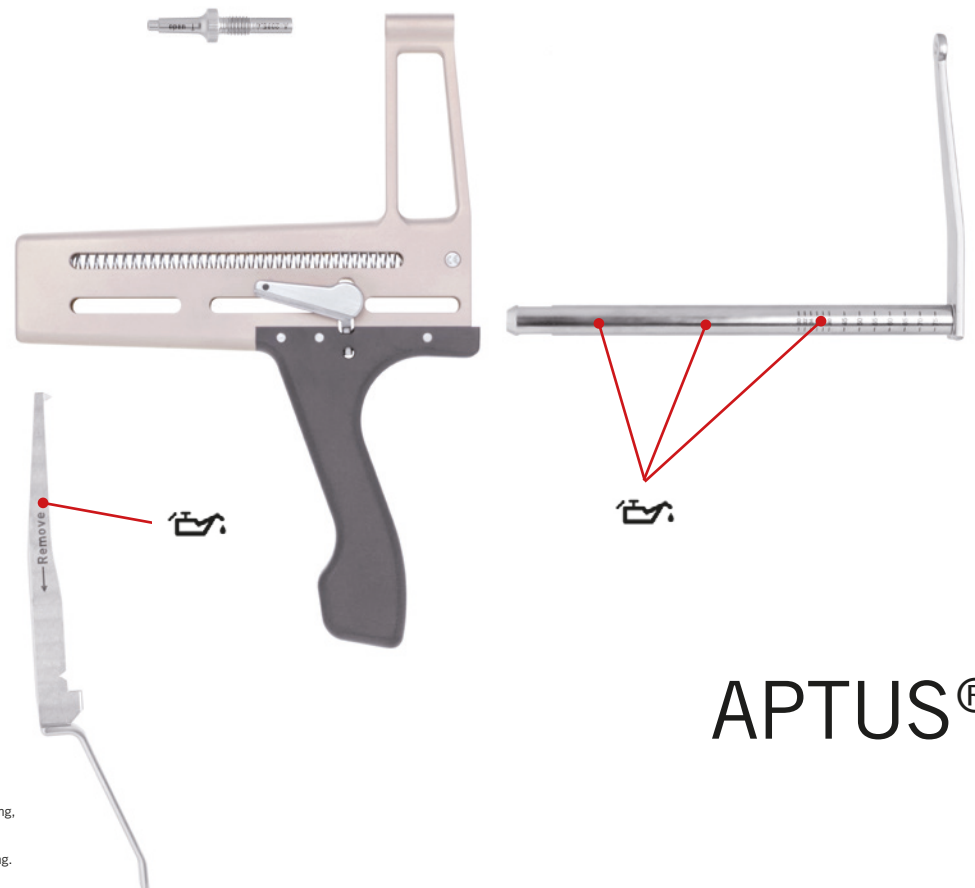
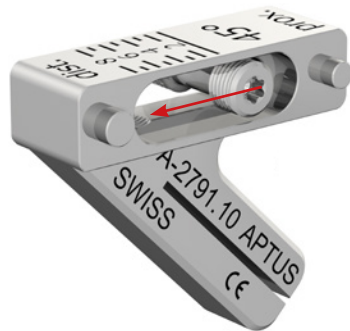
1


No disassembling is needed for cleaning, disinfection and sterilization of the articles A-2791.10 and A-2791.20. These processes can be done with the articles assembled.

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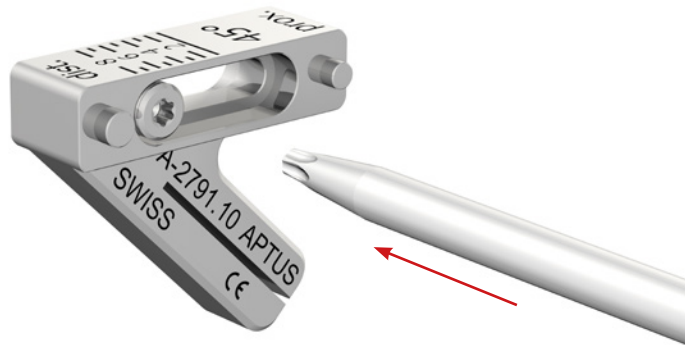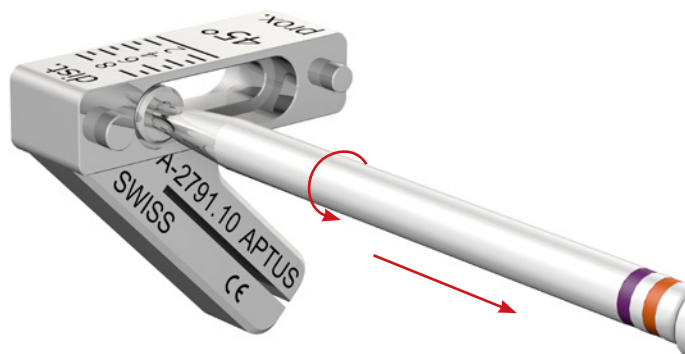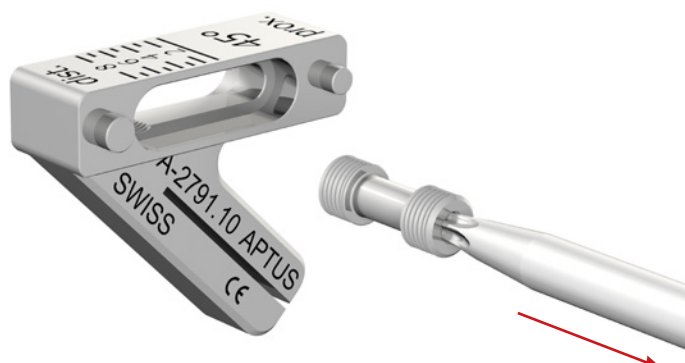
APTUS®

1

No disassembling is needed for cleaning, disinfection and sterilization of the article A-2791.30. These processes can be done with the article assembled.

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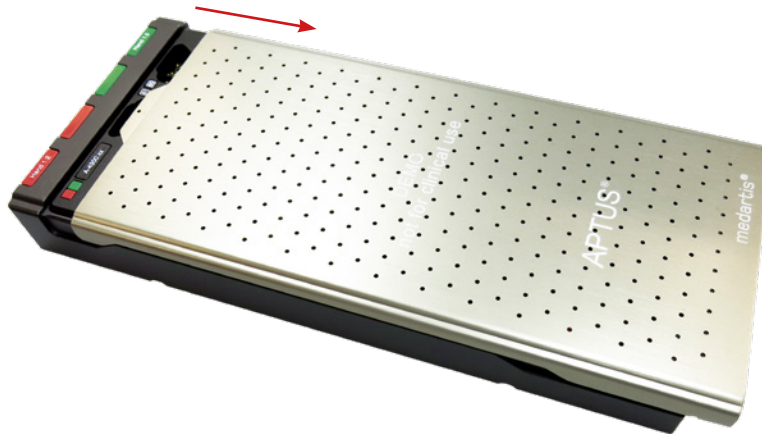
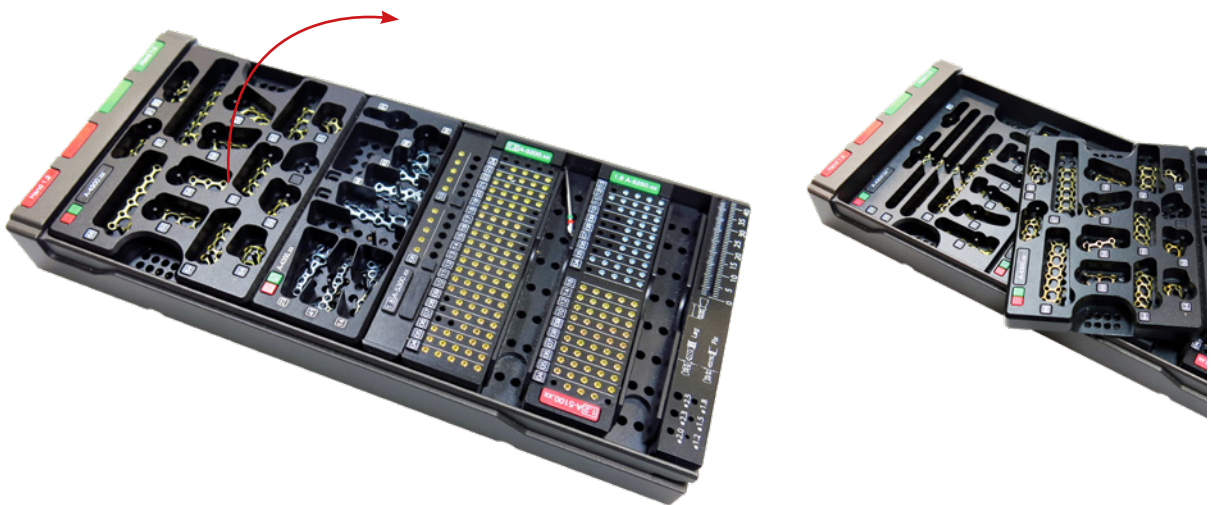
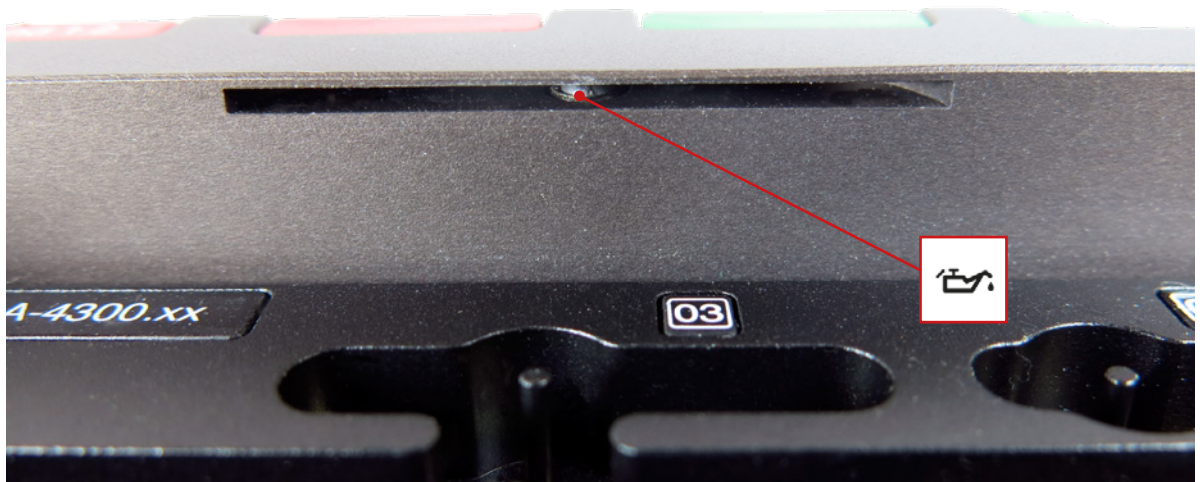
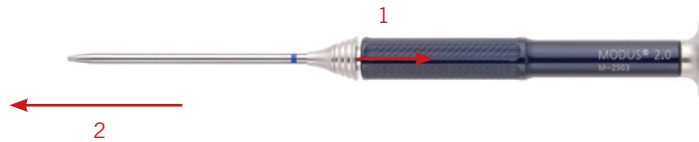
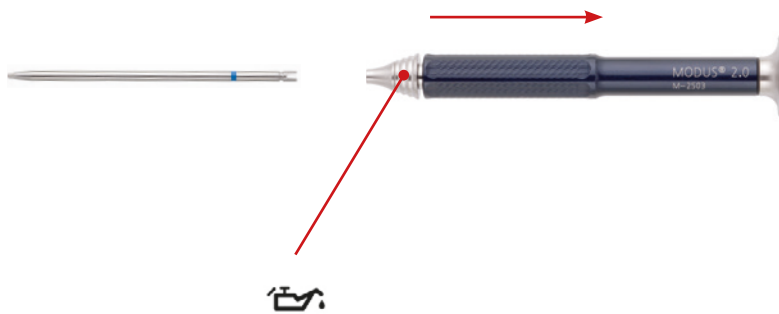
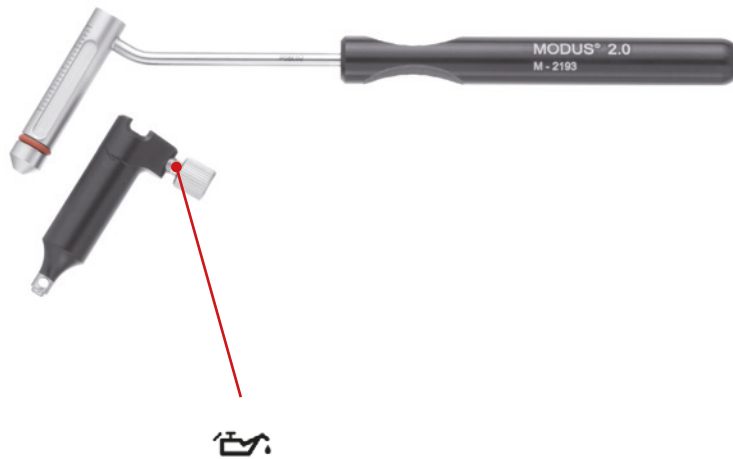
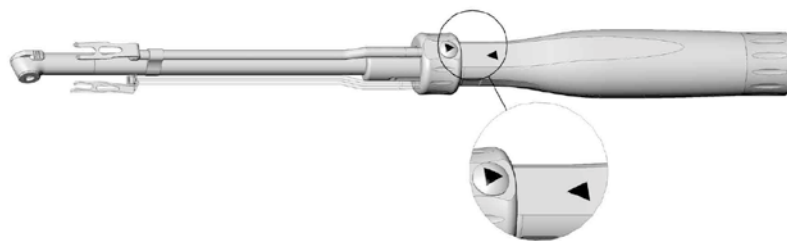
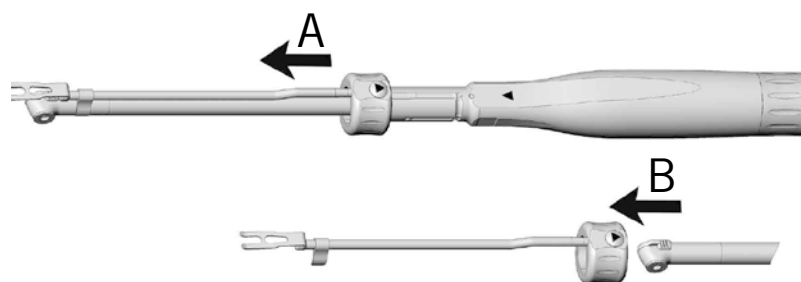
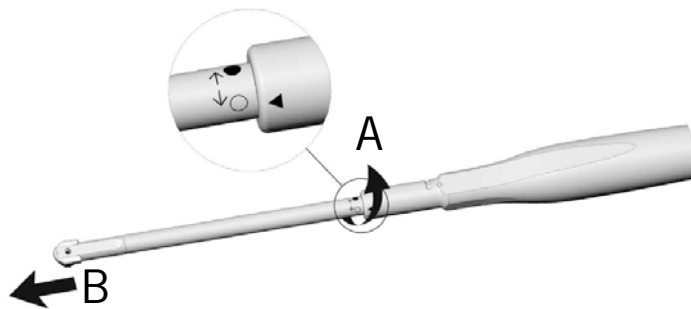
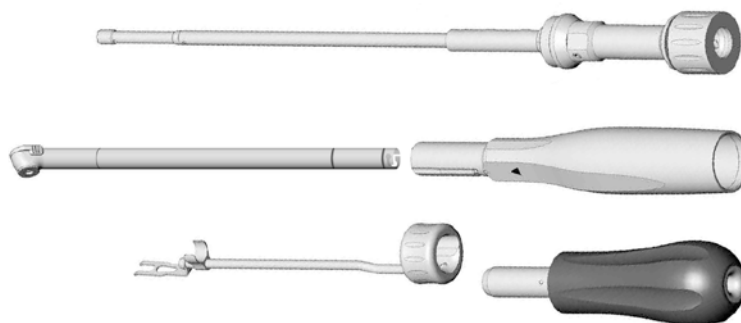
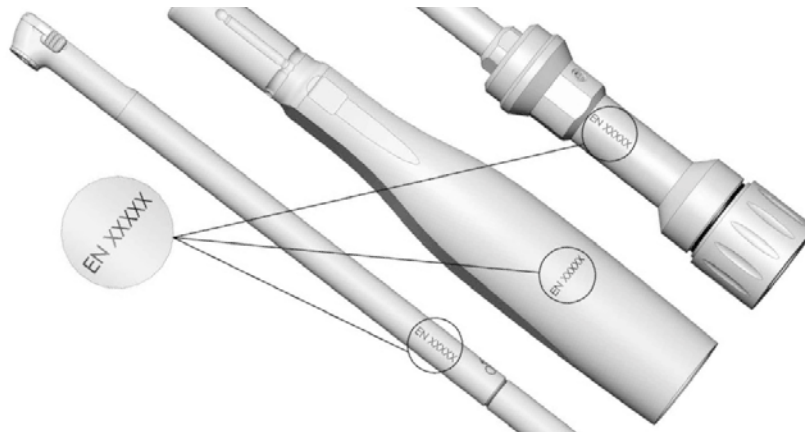
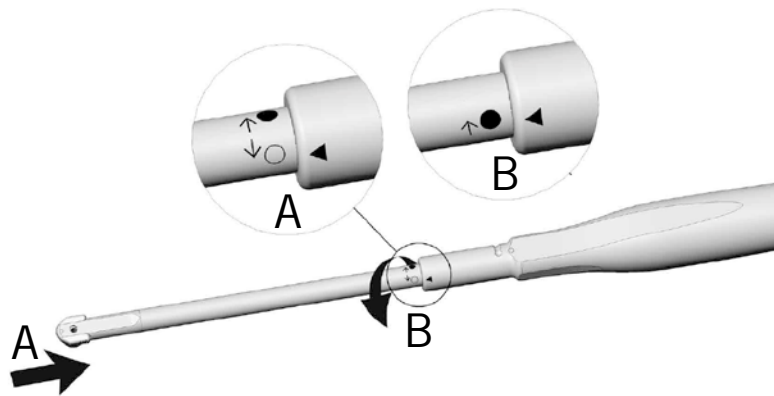
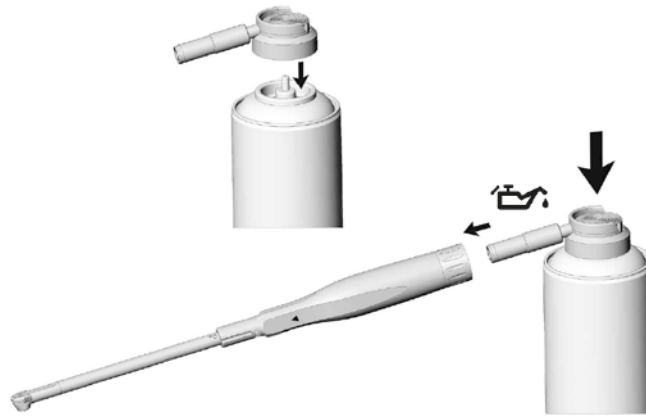
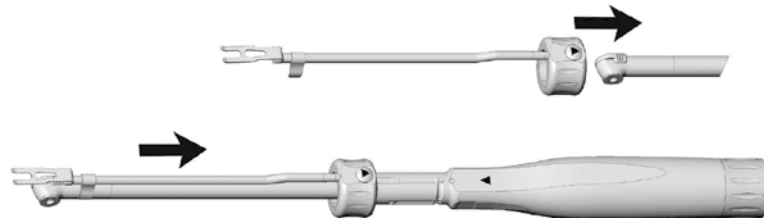
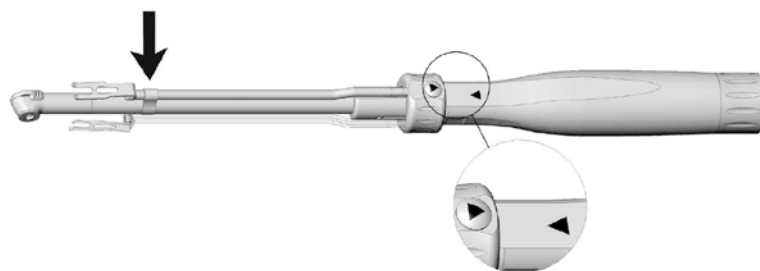
3

MODUS®

CORP-00000200_v7 / © 2019-05, Medartis AG, Switzerland. All technical data subject to alteration.

MANUFACTURER & HEADQUARTERS

Medartis AG | Hochbergerstrasse 60E | 4057 Basel/Switzerland
P +41 61 633 34 34 | F +41 61 633 34 00 | www.medartis.com

SUBSIDIARIES

Australia | Austria | Brazil | France | Germany | Japan | Mexico | New Zealand | Poland | UK | USA

For detailed information regarding our subsidiaries and distributors, please visit www.medartis.com

Disclaimer: This information is intended to demonstrate the Medartis portfolio of medical devices. A surgeon must always rely on her or his own professional clinical judgement when deciding whether to use a particular product when treating a particular patient. Medartis is not giving any medical advice. The devices may not be available in all countries due to registration and/or medical practices. For further questions, please contact your Medartis representative (www.medartis.com). This information contains CE-marked products.
For US only: Federal law restricts this device to sale by or on the order of a physician.